User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Neoflex: *Neo 10*

Contents

Preface	4
1. Your product	5
2. Before use	6
2.1. Intended use and indications	6
2.2. Safety instructions and applicable risks	7
2.3. Symbols on the product	8
3. Using your Neoflex system	9
3.1. Unfastening/Fastening the belts	9
4. Maintenance	10
4.1. Time of maintenance	10
4.2. Maintenance instructions	10
4.3. Expected service life	11
4.4. Reuse	11
4.5. End of use	11
4.6. Warranty	11
5. Technical specifications	12

Preface

Congratulations! You are now owner of a Vermeiren Neoflex belt!

Thanks for your trust in the products of Vermeiren. To support you on the use and operating options of your product, this manual is offered. If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. They will be glad to help you on this subject.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it according to the instructions in this manual.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.


Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	User manual of the wheelchair For user and specialist dealer
	EC declaration of conformity


1. Your product

EN


Neo 10.01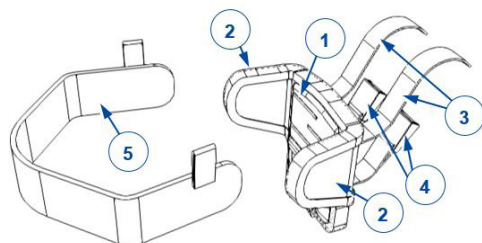

Neo 10.02

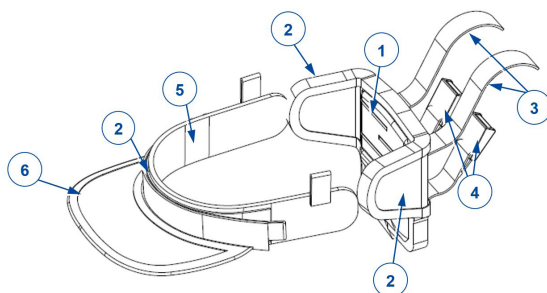Neo 10.02 Kids

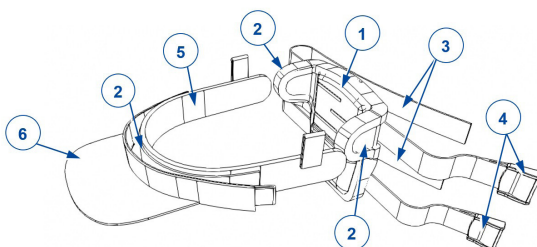1. Padded support
2. Hook-and-loop patch
3. Attachment and adjustment belt
4. Buckle
5. Retaining strap
6. Detachable visor

2. Before use

2.1. Intended use and indications

In this paragraph a brief description of the intended use of your product is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- This product is a medical device.
- Indications and contraindications: the Neoflex stabilisation belts are intended for people who require external support while sitting in order to maintain a correct position, or for those who want to stop the process of pathological positioning of the head, trunk, pelvis or lower limbs.
- The system should only be used by people with healthy skin, normal breathing, no mental disabilities, and no fractured bones in places where the Neoflex belts are touching the body. It should not be used for pregnant women.
- The Neo 10 systems are also not recommended for users with spasm attacks, or users who cannot bear to have the retaining strap on their forehead due to mental or sensory issues.
- The stabilisation belts are intended to be used on wheelchairs or other seating systems. They should NOT be used as a restraint or as a seat belt in a moving vehicle.
- Please read all technical details and limits of your Neoflex system in chapter 5..
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.2. Safety instructions and applicable risks

CAUTION

Risk of injuries and/or damage

- Please read and follow the instructions in this manual. Otherwise you may get injured, or your product may get damaged.
- This product should only be installed and fitted to the user by a qualified person.

Keep the following general warnings in mind during use:

- Only use the attachment systems provided by Vermeiren. Do not structurally modify the device to which Neoflex is attached.
- Before each use, check the product and attachments to the wheelchair/ chair to make sure they are undamaged and well secured.
- After each use, check the user's skin in the places of contact. Increasing redness and irritation can evolve into more serious injuries such as pressure ulcers.
- In case of damage, do not attempt to repair the product yourself. Contact your specialist dealer.
- Do not modify your product in any way.
- Accidental unfastening of the Neoflex system may cause the user to fall. For users with a cognitive or motion disorder that may lead to accidental release, an attendant or caregiver should be present when using the Neoflex system.
- It is not recommended to use the device when smoking.
- If the packaging of your product has been damaged, (unintentionally) opened, or affected by environmental conditions (moisture, heat,...) upon delivery, check the device integrity of your product. When in doubt, contact your specialist dealer.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For future communication, make sure to keep the LOT number from the packaging.

2.3. Symbols on the product

	Declaration of conformity
	Medical device
	Batch number
	Do not iron
	Machine-washable at 40 degrees and using the delicate programme
	Handwash
	Do not bleach
	Do not tumble dry
	Do not dry clean
	Manufacturer
	Attention: important information
	It is advised to read the manual

3. Using your Neoflex system

WARNING

Risk of injuries

- First read previous chapters and inform yourself about the intended use. Do NOT use your Neoflex system unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

3.1. Unfastening/Fastening the belts

CAUTION

Risk of injuries or damage

- The following actions can be done by the user, attendant or caretaker. All other adjustments are done by your specialist dealer.
- Make sure that the attendant or caregiver is aware of how to unfasten the Neoflex system in case of emergency.
- Make sure that your fingers, clothes, buckles or hair don't get trapped during adjustment.
- When placing the retaining strap, make sure it is positioned without pain or discomfort for the user:
 - it should not push/pull the eyebrows down/up;
 - it should not push on the user's ears or the legs of their glasses;
 - it should not be fitted too tight around the forehead (a slight forward movement should be possible with the flexibility of the strap).
- Make sure that the retaining strap of Neo 10.02 is placed with the hook-and-loop patch away from the forehead.

The retaining strap can be (un)fastened using the hook-and-loop patches (see §1.). Attach the strap in a comfortable manner so that it does not slide off the forehead.

The visor of Neo 10.02 can be attached using the hook-and-loop patch on the retaining strap.

4. Maintenance

4.1. Time of maintenance

CAUTION

Risk of injuries and damage

- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

4.1.1. Before each use

Inspect the following points:

- All parts (neoprene fabric, stitching, buckles, hook-and-loop patches, straps, mounting system): present, undamaged and showing no excessive wear, no discolouration.
- All parts: Clean, see § 4.2.1..

4.1.2. If stored

Make sure that your Neoflex system is stored in a dry environment to prevent mould from growing, or the system from being damaged.

4.2. Maintenance instructions

4.2.1. Cleaning

CAUTION

Risk of damage

- Your product is suitable for cleaning by hand wash, or by using the delicate cleaning programme on your washing machine.
- Do not dry clean, bleach or dye your product.

Minor dirt can be cleaned with a cloth. Thorough cleaning by hand wash or a delicate washing programme is advisable depending on the use frequency and the user's needs.

The upholstery can be washed with lukewarm water (max. 40°C) and a mild soap. Do not use abrasive cleaning agents to clean.

Let the Neoflex system dry thoroughly on a flat surface before using. Do not dry the Neoflex system over intense heat sources (e.g. radiators or stoves) or in direct sunlight.

4.3. Expected service life

The average service life of your Neoflex system is 5 years. Depending on the frequency of use and maintenance, the service life of your Neoflex product will increase or decrease.

4.4. Reuse

Neoflex products are meant for single patient multiple use only.

4.5. End of use



At end of life, you need to dispose of your Neoflex system according to the local environmental legislation.

4.6. Warranty


The warranty on this product is subject to the general terms and conditions of each country.

5. Technical specifications

The technical details below are only valid for the Neoflex systems at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your system has been modified, damaged, or is severely worn.

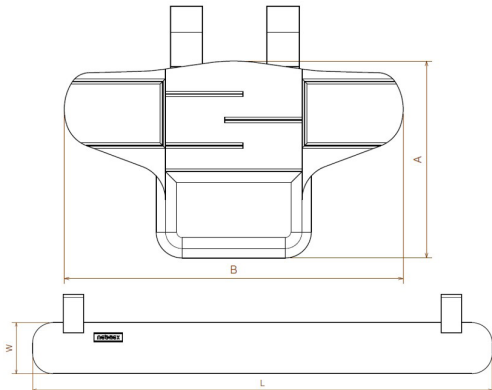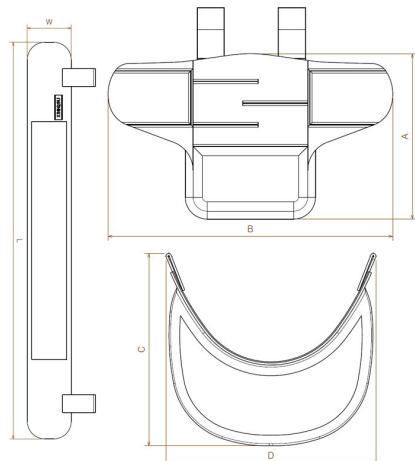Brand	Vermeiren					
Product group	Neoflex					
Type	Neo 10					
Description	Dimensions (mm)					
Neo 10.01						
		A	B	W	L	
	M	146	265	45	400	
	L	146	265	45	450	
Neo 10.02						
		A	B	W	L	D
	M	146	265	45	400	140
	L	146	265	45	450	140

Neo 10.02 Kids

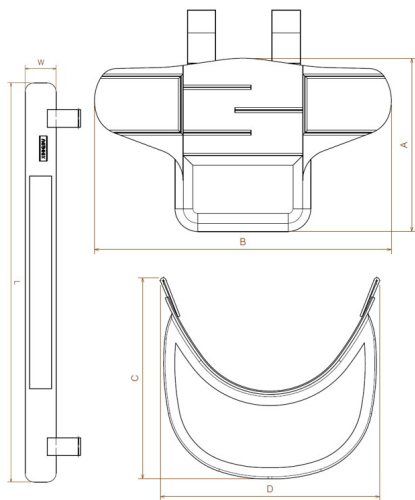One
size

A

B

W

L

D

140

210

35

450

110

Use temperature

+5°C - +41°C

Storage and use humidity

30% - 70%

We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°

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**Service registration form**

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

VERMEIREN GROUP

Vermeirenplein 1/15

2920 Kalmthout

Belgium

www.vermeiren.com

Version: A, 2025-06

Basic UDI-DI: 5415174180939Neoflex2H

All rights reserved, including translation.

VERMEIREN
we care **for** you