
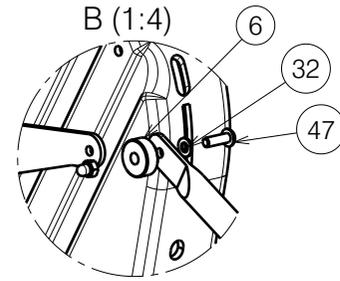


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

EDEN			
Scale: 1:8	Drafted	Verified	Confirmed
Date	15/10/2018	17/12/2025	17/12/2025
Initials	DR	PM	AZ

-

Plan No.
EP00338/G

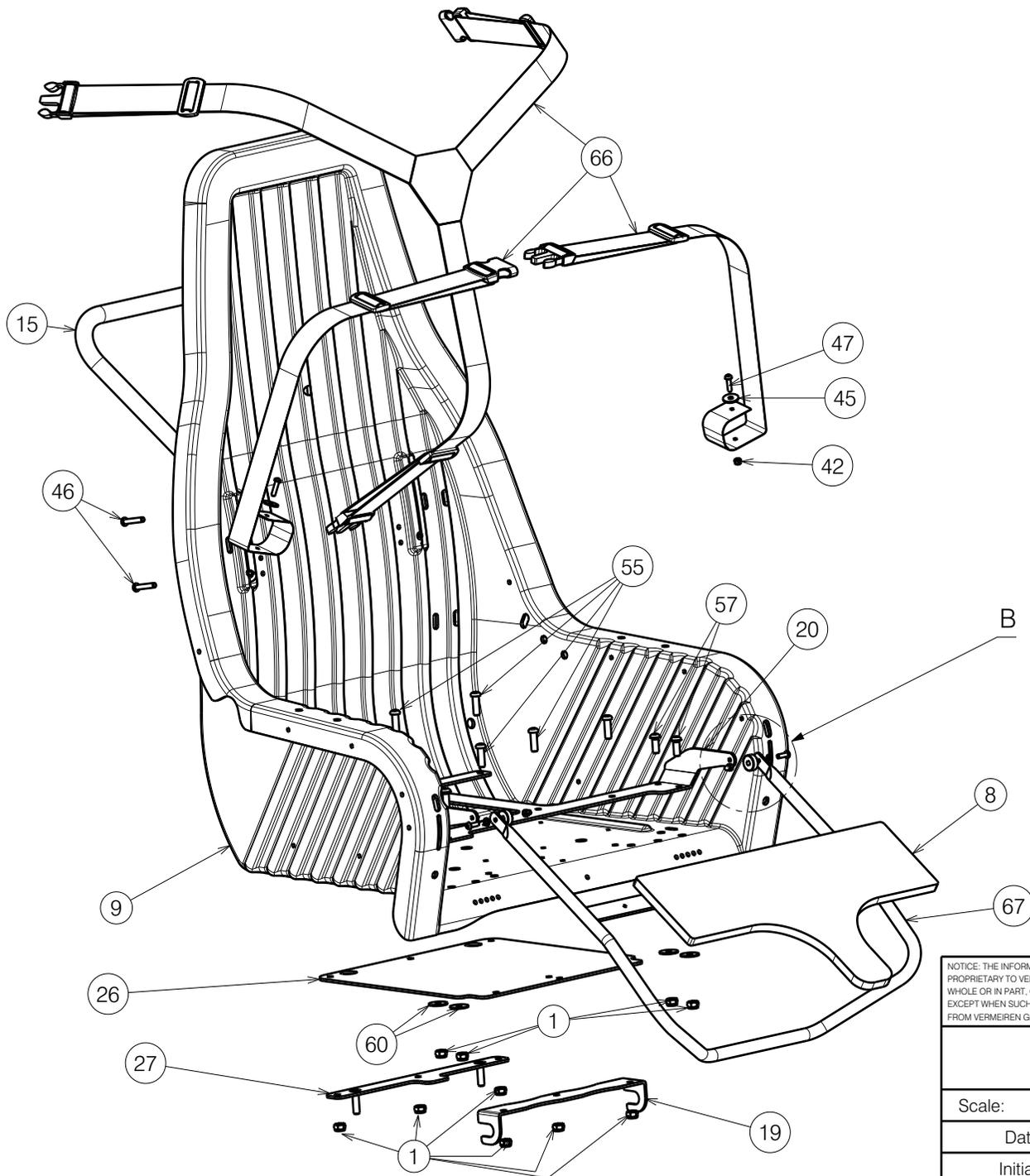


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.


EDEN			
Scale: 1:8	Drafted	Verified	Confirmed
Date	15/10/2018	17/12/2025	17/12/2025
Initials	DR	PM	AZ

-
Plan No.
EP00338/G

Upper position:

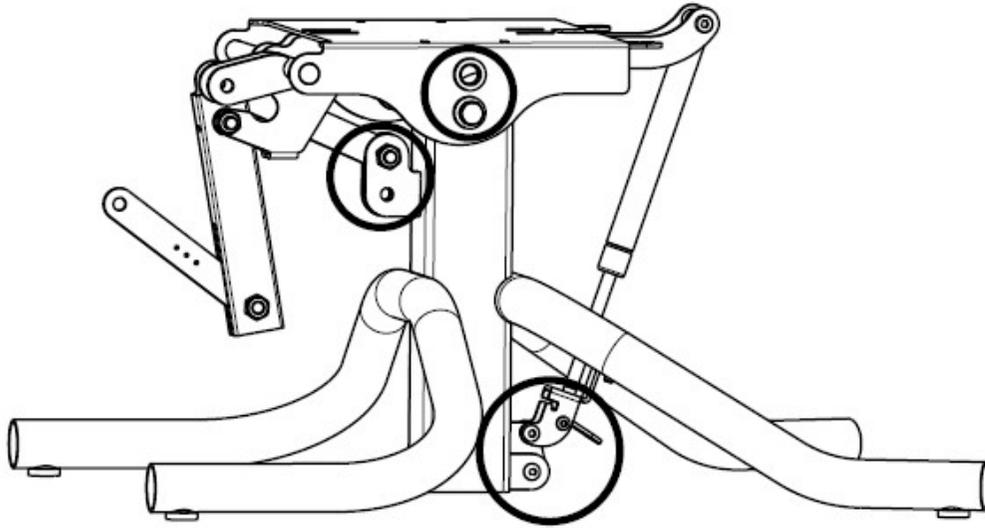

Lower position:

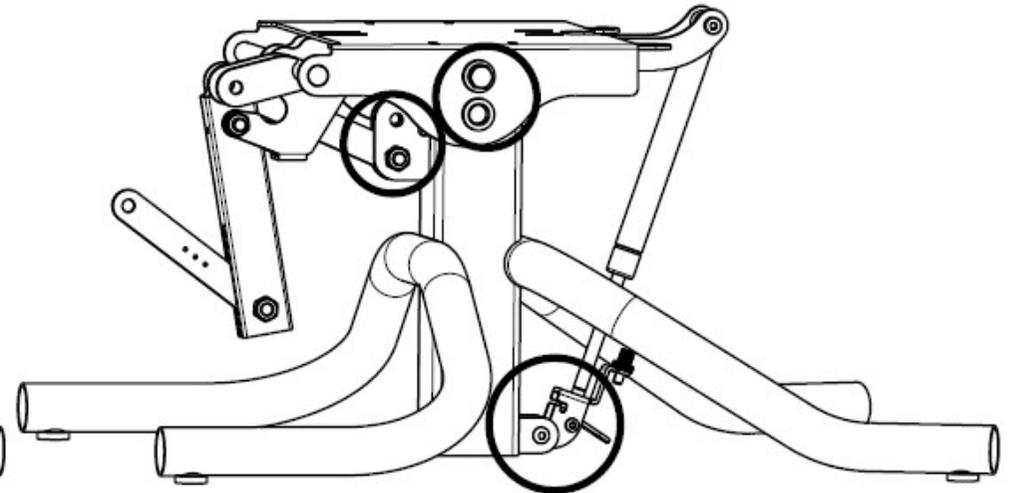

Sheet 3/4

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

EDEN			
Scale:	1:8	Drafted	Verified
Date	15/10/2018	17/12/2025	17/12/2025
Initials	DR	PM	AZ

-
Plan No.
EP00338/G

Bill of Material				Bill of Material				Bill of Material				
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	
1	X001016	1001682	13	32		Washer M5 DIN125	1000972	61		Brake handle	1996806	1
2	X002029	1991841	4	33		M10x50 DIN933	1001453	62		Gas spring	1996871	1
3	X002708	1996783	2	34		Winget Nut M8	1001517	63		-	-	-
4	X003646	1999963	1	35		Washer M8 DIN125	1001556	64		-	-	-
5	X005000	1767900	2	36		Washer M10 DIN125	1001557	65		M10x30	2991162	3
6	XP00099 (38)	2990067	2	37		Washer M10 DIN127B	1001566	66		Set of belts	2991544	1
7	XP00782	1909147	1	38		Washer M8 DIN127A	1001570					
8	XP01034 (48)	HP00196	1	39		Rivet 4,8x17	1001586					
	XP01078 (44)	HP00210	1	40		Tube Cap D25	1001608					
	XP01061 (38)	HP00204	1	41		Tube Cap D38	1001611					
9	XP01250	2992293	1	42		Nut M5 DIN985	1001679					
10	XP01316	2894400	1	43		Nut M6 DIN985	1001680					
11	XP01318	2894100	1	44		Nut M10 DIN985	1001683					
12	XP01327	2895700	1	45		Washer M5 DIN9021	1005138					
13	Y011652	1655254	1	46		M6x35	1006057					
14	Y011660	1655554	1	47		M5x16 (38/44,48)	1006112					
15	YP00541	18216	1			M5x20 (38)	1998368					
16	YP04047	28457	1	48		M8x45	1994473					
17	YP04048	28456	1	49		M10x35	1009442					
18	YP04051 (48)	28459	1	50		M8x60	1997781					
	YP04412 (44)	28821	1	51		M10x53	1551600					
	YP04330 (38)	28815	1	52		Regulating Screw M7	1900008					
19	YP04061	2846700	1	53		M8x50	1902483					
20	YP04079 (48)	28468	1	54		Wheels 50mm	1908666					
	YP08251 (44)	26120	1	55		M8x25	1990501					
	YP04415 (38)	28822	1	56		M5x5 DIN916	1991323					
21	YP04088	28455	1	57		M8x20	1992349					
22	X005136	1910092	2	58		Pin 8x40	1992510					
23	YP04164	28470	1	59		Nut M5 DIN986	1992804					
24	YP04168	28471	1	60		Washer M8 DIN9021	1996574					
25	-	-	-									
26	YP04343	28816	1									
27	YP04386	28519	1									
28	YP06666	27330	1									
29	YP06667	27331	1									
30	YP06670	27332	1									
31	Pin 6x26	G040500	1									

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

EDEN			
Scale: 1:8	Drafted	Verified	Confirmed
Date	15/10/2018	17/12/2025	17/12/2025
Initials	DR	PM	AZ

-
Plan No.
EP00338/G
