



Detail A  
Scale: 2:5

E001140 / A


<b>SIGMA BACK FRAME L44 SE34</b> Sheet 1/2				Plan No. E001140 / A
Scale: 1:5	Drafted	Verified	Confirmed	
Date	18/11/2019	18/11/2019	18/11/2019	
Initials	SD	SD	PJ	

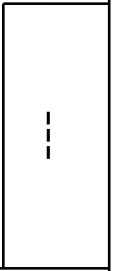

Material			
No.	Plan	Article	Quantity
1	Y022312	2049854 (40)	1
	Y022980	2049754 (45)	1
	Y022313	2049954 (50)	1
2	Y023126	2053954 (40)	1
	Y023127	2054054 (45)	1
	Y023128	2054154 (50)	1
3	X005471	2011600	1
4	M8x16 DIN912	1001498	4
5	X001119	1001554	4
6	M8 DIN125	1001556	2
7	M6x30 DIN7984	1992721	1
8	M8 DIN985	1001682	5
9	Ø8x40	1992510	1
10	X000862	1006463	4
11	M8x40 ISO7380	1008739	1
12	Y018564	1774200	2
13	M8x50 ISO7380	1902483	2
14	X004888	1906839 (40)	1
	X004857	1906740 (45)	1
	X004889	1906841 (50)	1
15	Y021512	2011700	1
16	Y021526	2011554	1
17	Y021513	2011454	1
18	---	1906843 (40)	1
	---	1906844 (45)	1
	---	1906845 (50)	1
19	25x25	1001617	2
20	M5x10	1911896	1
21	20x20	1001612	1
22	Ø22	1990648	4
24	M8x20 ISO7380	1992349	2
25	30x20	1997922	2
26	---	2993352	1

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

E001140 / A

SIGMA BACK FRAME L44 SE34				Plan No.
Sheet 2/2				E001140 / A
Scale: 1:5	Drafted	Verified	Confirmed	
Date	18/11/2019	18/11/2019	18/11/2019	
Initials	SD	SD	PJ	