


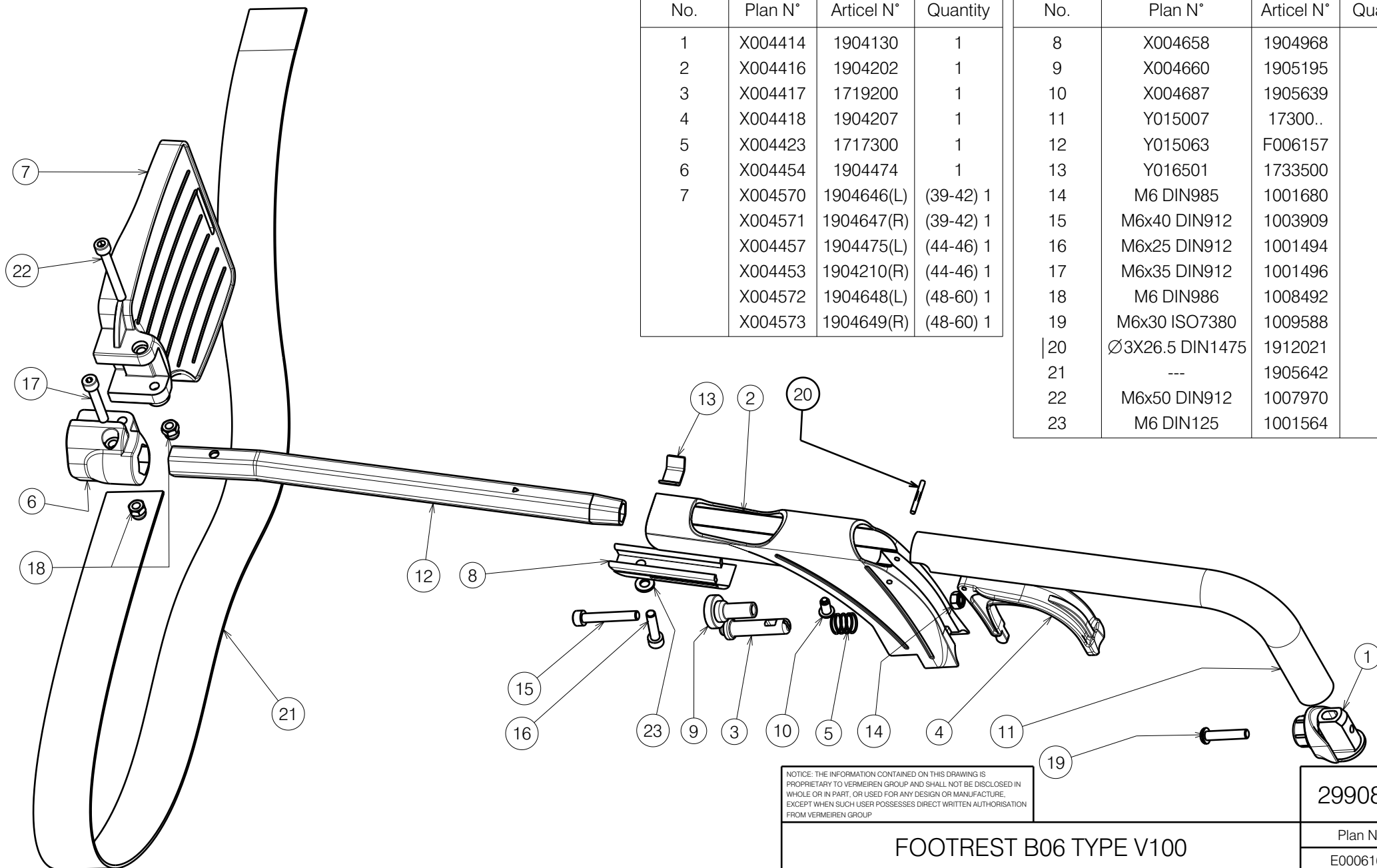

Material			
No.	Plan N°	Articel N°	Quantity
1	X004414	1904130	1
2	X004416	1904202	1
3	X004417	1719200	1
4	X004418	1904207	1
5	X004423	1717300	1
6	X004454	1904474	1
7	X004570	1904646(L)	(39-42) 1
	X004571	1904647(R)	(39-42) 1
	X004457	1904475(L)	(44-46) 1
	X004453	1904210(R)	(44-46) 1
	X004572	1904648(L)	(48-60) 1
	X004573	1904649(R)	(48-60) 1

Material			
No.	Plan N°	Articel N°	Quantity
8	X004658	1904968	1
9	X004660	1905195	1
10	X004687	1905639	1
11	Y015007	17300..	1
12	Y015063	F006157	1
13	Y016501	1733500	1
14	M6 DIN985	1001680	1
15	M6x40 DIN912	1003909	1
16	M6x25 DIN912	1001494	1
17	M6x35 DIN912	1001496	1
18	M6 DIN986	1008492	2
19	M6x30 ISO7380	1009588	1
20	Ø3X26.5 DIN1475	1912021	1
21	---	1905642	1
22	M6x50 DIN912	1007970	1
23	M6 DIN125	1001564	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

2990874



FOOTREST B06 TYPE V100

Plan No.

E000616/K

Scale: 3:10	Drafted	Verified	Confirmed
Date	20-04-2010	08-07-2025	08-07-2025
Initials	TM	MP	SD



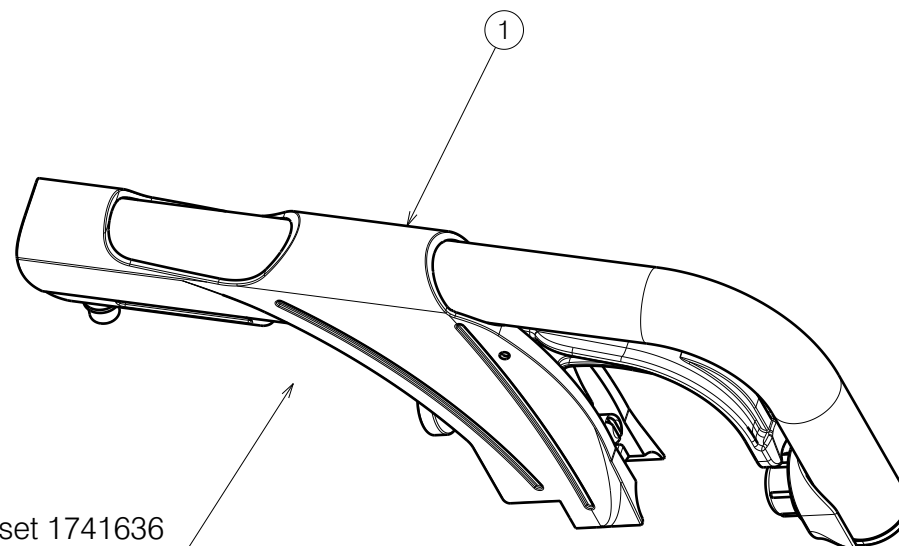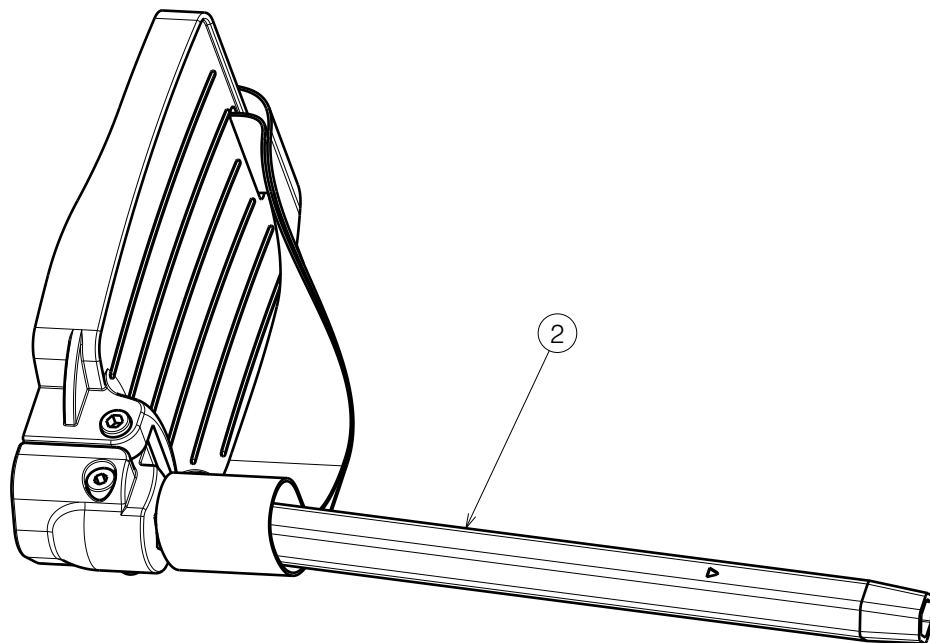

For assembly in PL: use set 1741636

BOM

No.	Plan N°	Articel N°	Quantity
1	Butt set	17416..	2
2	Size 39-42	1906150	1
	Size 44-46	1906151	1
	Size 48-50	1906152	1

Quantities are for one pair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

2990874

FOOTREST B06 TYPE V100

Plan No.

E000616/K

Scale: 3:10

Drafted

Verified

Confirmed

Date

20-04-2010

08-07-2025

08-07-2025

Initials

TM

MP

SD

