


Glue with Weicon
AN 302-43
(H900009)

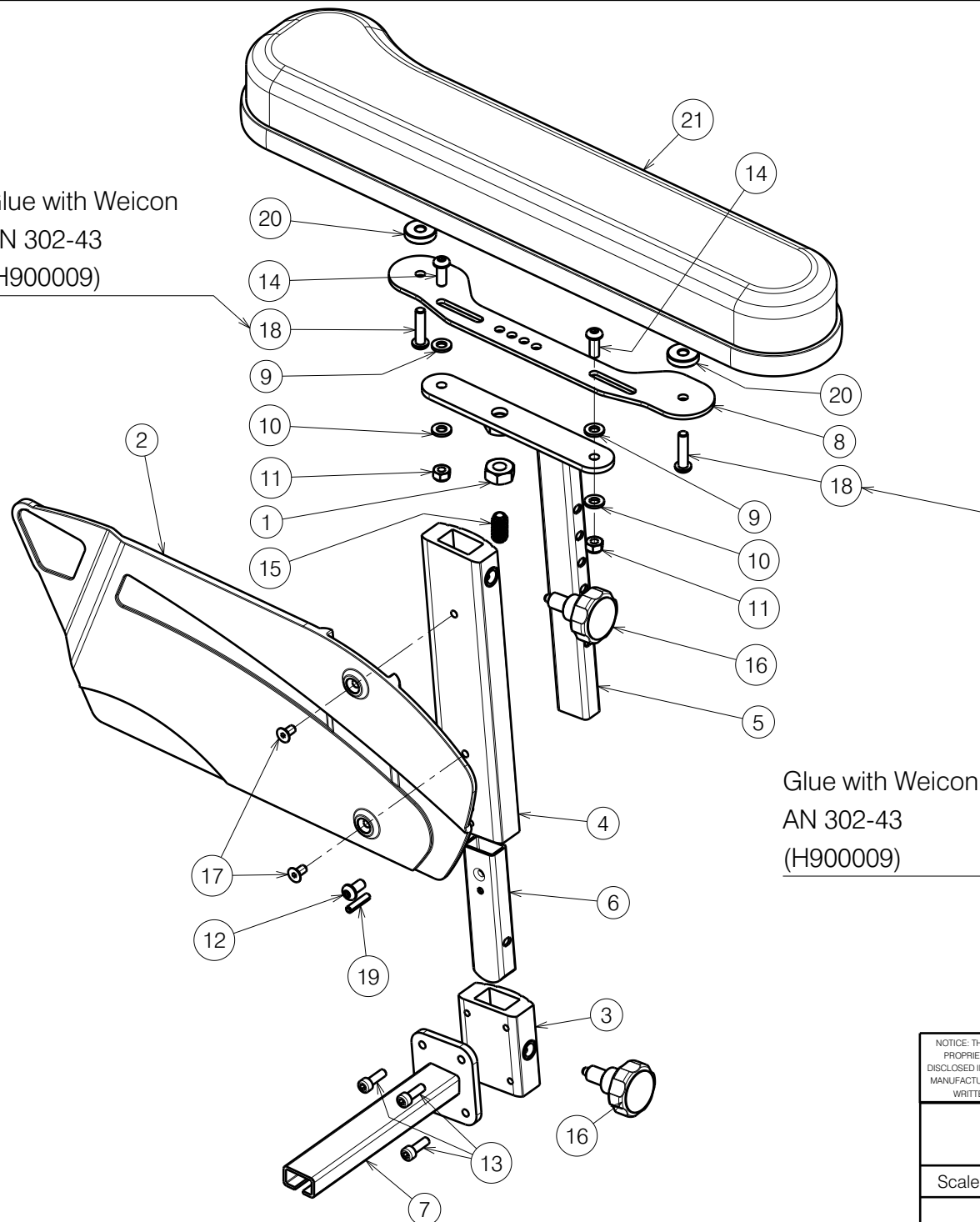

Glue with Weicon
AN 302-43
(H900009)

Material			
No.	Plan	Article	Quantity
1	M10 DIN934	1001672	2
2	X003347	1900204 (L)	1
	X003348	1900205 (R)	1
3	X004812	2051654	2
4	X004813	2051754 (L)	1
	X004814	2051854 (R)	1
5	Y008789	1659301	2
6	Y011205	1657901	2
7	Y011207	1658000 (L)	1
	Y011750	1662500 (R)	1
8	Y011751	1662654	2
9	M6 PVC DIN125A	1000973	4
10	M6 DIN125A	1001564	4
11	M6 DIN985	1001680	4
12	M6x12 Black	3992768	2
13	M5x16 DIN912 Black	1915415	8
14	M6x16	1008459	4
15	M10x19	1991432	2
16	Ø35 M10x1	1996819	4
17	M5x10 DIN7991/8.8	1999977	4
18	M6x25 Black	2994062	4
19	Ø4x20 DIN1481	1009584	2
20	XP00227	2990147	4
21	Armrest L Black	2995547 (L)	1
	Armrest R Black	2995548 (R)	1

Only left side depicted / Quantity for 1 pair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE
DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR
MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT
WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

INOVYS 2 EVO B05 ARMREST				-
				Plan No. E00036770/D1.1
Scale: 1:4	Drafted	Verified	Confirmed	
Date	22/06/2022	26/05/2026	26/05/2026	
Initials	DR	PM	AS	