


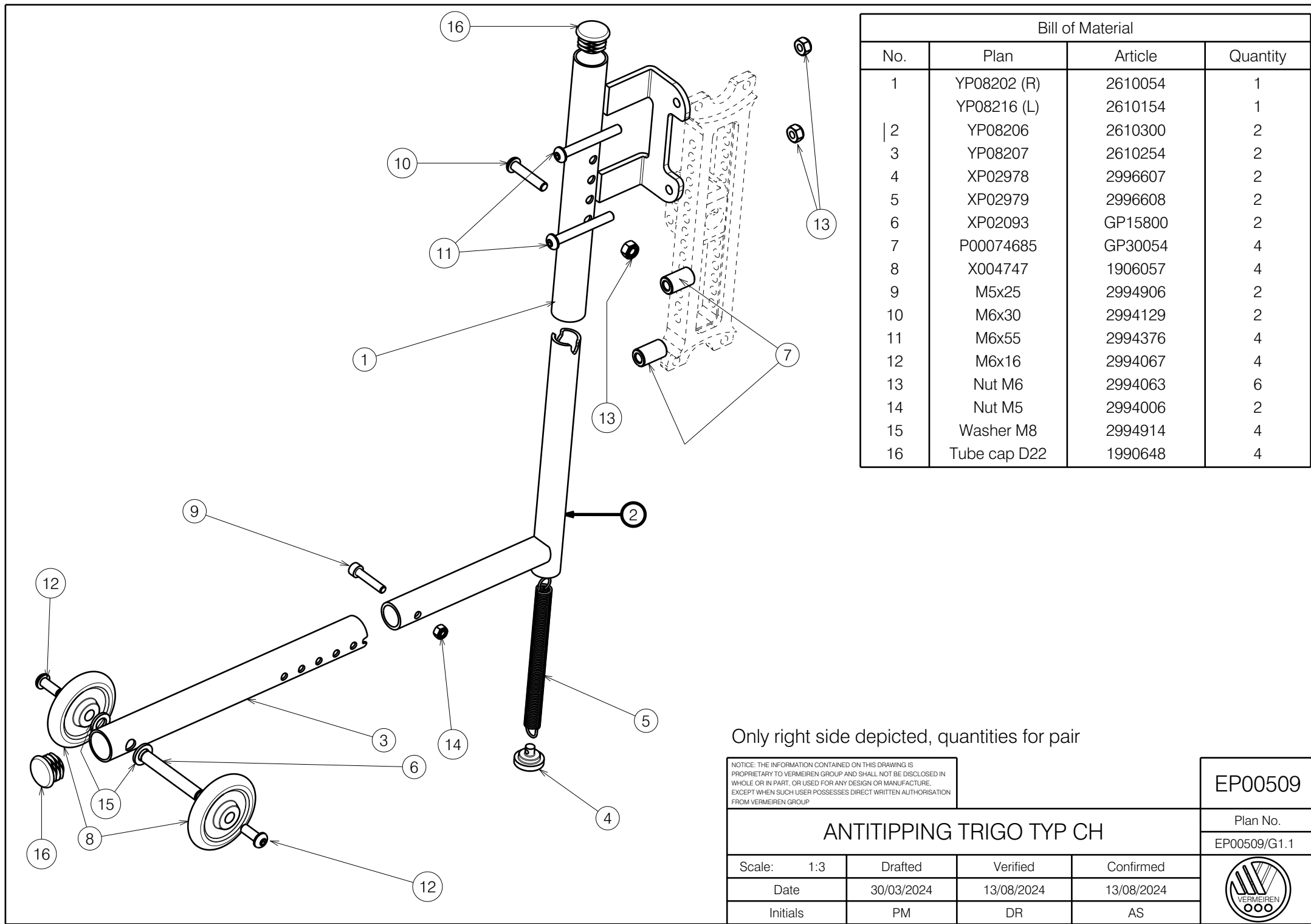

Bill of Material			
No.	Plan	Article	Quantity
1	YP08202 (R)	2610054	1
	YP08216 (L)	2610154	1
2	YP08206	2610300	2
3	YP08207	2610254	2
4	XP02978	2996607	2
5	XP02979	2996608	2
6	XP02093	GP15800	2
7	P00074685	GP30054	4
8	X004747	1906057	4
9	M5x25	2994906	2
10	M6x30	2994129	2
11	M6x55	2994376	4
12	M6x16	2994067	4
13	Nut M6	2994063	6
14	Nut M5	2994006	2
15	Washer M8	2994914	4
16	Tube cap D22	1990648	4

Only right side depicted, quantities for pair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

ANTITIPPING TRIGO TYP CH				EP00509
Scale: 1:3				Plan No.
Date				EP00509/G1.1
Initials				
Drafted	30/03/2024	Verified	13/08/2024	
Confirmed	30/03/2024	Confirmed	13/08/2024	
PM	DR	AS		