


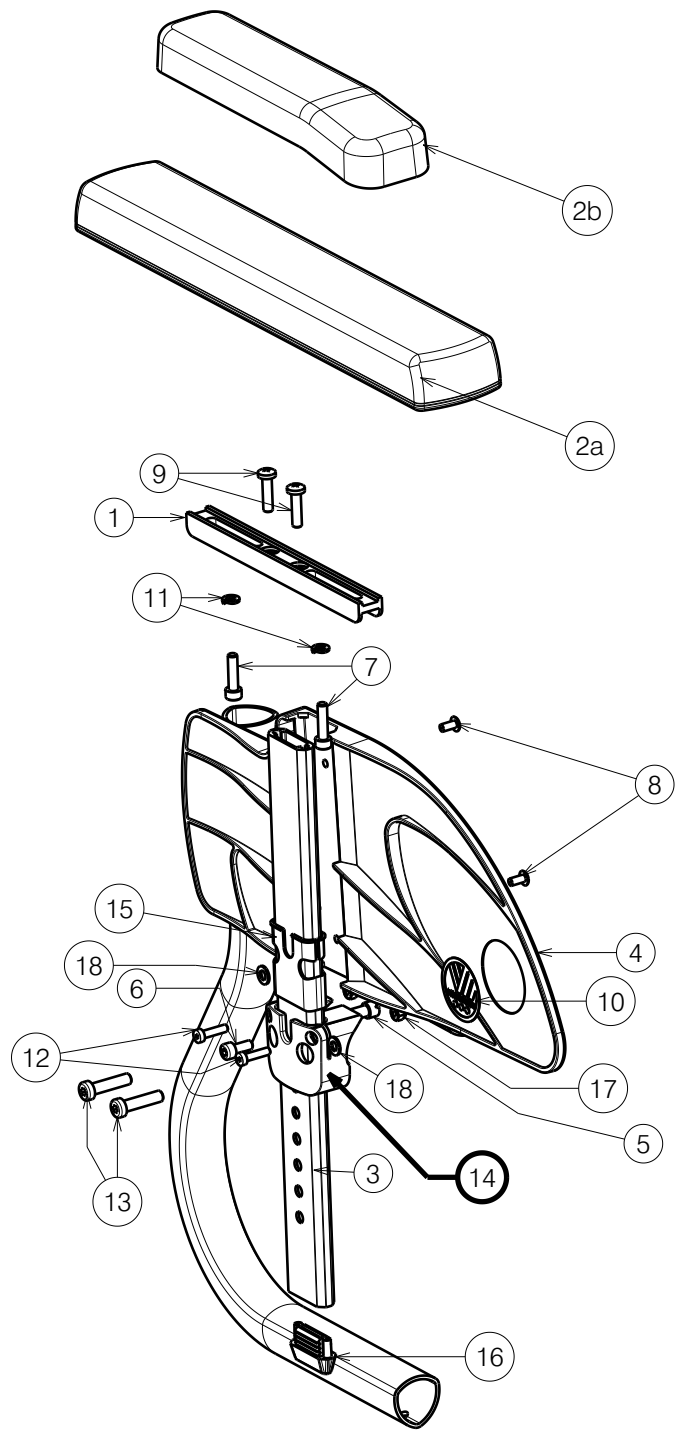

Material			
No.	Plan	Article	Quantity
1	X005517	2014999	1
2a	X005519	1911778	1
2b	XP03265 (short)	2996861	1
3	X005561	2021699	1
4	X005566	1912285 (L)	1
	X005567	1912286 (R)	1
5	XP02501	GP18200	1
6	M5x12 DIN912	2994072	1
7	M5x20 DIN912	1003907	2
8	D4x8	1992805	2
9	M5x16 DIN7985	3992507	2
10	---	T001002	1
11	M5 DIN127	1906561	2
12	M4x16	2994070	2
13	M6x25	2994897	2
14	XP03335	GP27426	1
15	XP03091	2996662	1
16	XP03080	2996656	1
17	M4	1001678	2
18	M5DIN125	2996010	2

| REMOVABLE SIDE GUARD NOT POSSIBLE WITH CAMBER 6

EP00393 / I

V500 BI5 FOR TRIGO				Plan No.
				EP00393/ I
Scale: 1:4	Drafted	Verified	Confirmed	
Date	18/04/2019	22/09/2025	22/09/2025	
Initials	FD	PM	AS	