VERMEIREN

Magda

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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EN Contents

1. Technical details	2
2. Components	2
3. Explanation of symbols	2
4. Check after delivery	2
5. Intended use	2
6. Before use	3
7. Assembly of the swivel seat	3
8. Use of the swivel seat	3
8.1. Mounting the swivel seat to the bathtub	3
8.2. Removing the swivel seat from the bathtub	3
8.3. Turning the swivel seat	3
8.4. Transfer to/from swivel seat	3
9. Carrying the swivel seat	4
10. Safety instructions	4
11. Maintenance, inspection, cleaning	4

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INSTRUCTION MANUAL
Swivel bath transfer seat
"Magda"

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Swivel bath transfer seat
Model	Magda
Width	535 mm
Length	735 mm - 785 mm
Height	530 mm
Seat width	440 mm
Seat depth	360 mm
Backrest height	370 mm
Inner width bathtub	560 - 660 mm
Weight total	4,20 kg
Patient weight	max. 130 kg
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the seat, read the instruction manual carefully: it will familiarise you with this product.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

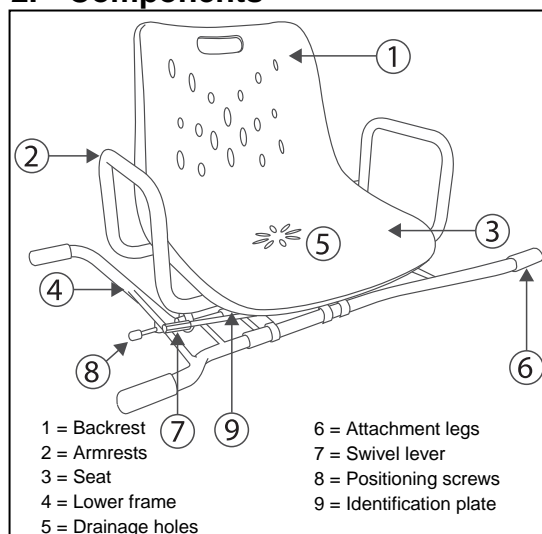
Fig 1. Components Swivel seat

3. Explanation of symbols

Maximum mass

CE conformity

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included (refer also to the assembly drawings):

- Swivel bath transfer seat
- Armrests
- Lower frame
- Positioning screws
- Clips
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

The seat shall be used by 1 person.

The seat shall be used indoor, in a bathtub.

Is your bathtub too small for a conventional tub / shower safety / transfer seat? Then the swivel bath transfer seat is the solution. The seat is ideal for people who feel insecure to sit on a bath board. The swivel seat can be used for a bathtub with inner width of 560 until 660 mm.

The swivel bath transfer seat is a 360° turning bath seat that allows easy transfer over the bath board. The seat can be locked into four 90 degree positions. The user sits on the seat from the side of the bath and then swivels over the bath for a wash or shower.

The construction of the seat allows full use by elderly people or during revalidation in home environment.

The seat shall not be used as toy for children. The seat should only be used on horizontal, non smooth surfaces of the bathtub board. The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

6. Before use

- Remove all packaging before using the swivel seat for the first time.
- Assembly of swivel seat.

7. Assembly of the swivel seat

Refer to the attached assembly drawings.

⚠ CAUTION: Risk of injury - Be careful you mount the legs of the lower frame in the correct manner. The legs must be pointed upwards.

8. Use of the swivel seat

The swivel bath transfer seat is ideal for usage in a bathtub. The specially designed rubbers at the end of each attachment leg will create a safe stability for the seat and avoid the sliding over the bath board.

The seat can be mounted on the top of the bathtub and can be easily removed when not in use. So other persons can use the bath.

8.1. Mounting the swivel seat to the bathtub:

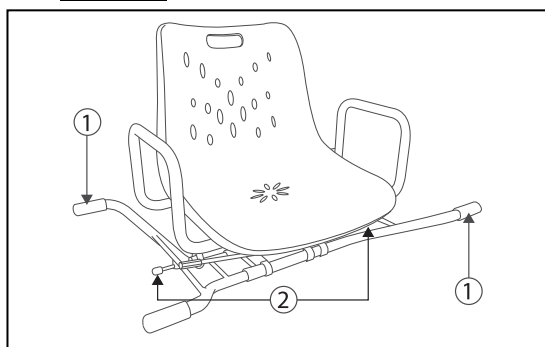
Fig 2. Mounting / removing swivel seat to the bath

⚠ CAUTION: Risk of injury - Check that the seat is properly fixed in the bathtub with the positioning screws ②.

⚠ CAUTION: Risk of injury - Check that at least 30 mm of each attachment leg ① rest securely on each side of the bath.

1. Place the swivel seat with the four attachment legs ① on the board of the bathtub.
2. Turn the positioning screws ② on both sides against the bath.
3. Check that the swivel chair is properly fixated to the bath.

8.2. Removing the swivel seat from the bathtub:

1. Loosen the positioning screws ② from the bath.
2. Take the swivel seat out the bath.

8.3. Turning the swivel seat:

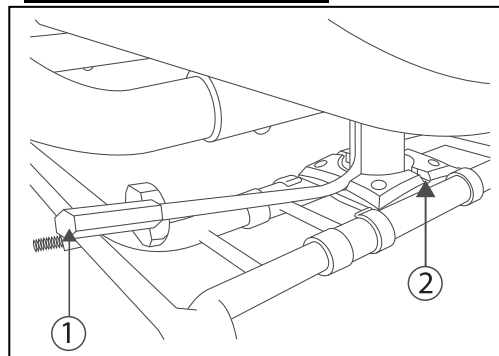
Fig 3. Turning the swivel seat

⚠ CAUTION: Risk of injury - Check that the seat is properly locked after turning.

Let someone assist you for a smoothly, safely turning of the swivel bath transfer seat.

1. Pull the swivel lever ① up. The lever comes out the locking groove ②.
2. Lift the legs of the patient up when turning to the washing position.
3. Turn the seat to the desired position. The seat can be locked in four 90° positions by placing the lever in one of the four locking grooves ②.
Desired positions:
 - **Patient position for washing:** place seat parallel with lower frame
 - **Patient position for transfer to/from swivel seat:** place seat perpendicular with lower frame
4. Lock the seat by placing the swivel lever ① in the correct locking groove ②.
5. Check that the seat is properly locked.

8.4. Transfer to/from swivel seat:

⚠ CAUTION: Risk of injury - Be careful that you not slip out on the bathroom floor.

For people who can walk:

1. Walk to the swivel seat.
2. Turn the swivel seat in a good position, it must be easy to transfer to the seat. (seat perpendicular with the lower frame)
3. Place yourself in the seat. For extra support you can grasp the armrests. **Be careful that you not slip out on the floor.**
4. Turn the swivel seat to the washing position.

For persons who has not the walking ability:

1. Let someone assist you to transfer to / from the swivel seat.
2. Turn the seat in the good position (seat perpendicular with the lower frame).
3. Lift the patient in the swivel seat. For extra support the patient can grasp the armrests.
4. Turn the swivel seat to the washing position.

When the swivel seat is placed too high for performing the transfer on a safe manner, you can make use of a step stool/bench.

9. Carrying the swivel seat

The best way to carry the swivel seat is to grasp the seat with the armrests.

10. Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for your seat is 130 kg.**
- Use the swivel seat on horizontal, non-smooth surfaces of the bathtub board.
- Keep it away from children. The swivel seat shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Risk of burns – Be careful when using in extreme hot or cold environments for a

sufficient amount of time and when touching.

- Keep fingers, cloths, jewelry or items away from moving parts of the swivel seat.
- Check that the seat is properly locked after turning.
- Check that the seat is properly fixed in the bathtub with the positioning screws.
- Check that at least 30 mm of each attachment leg rest securely on each side of the bath.
- Do not use the swivel seat when it is not working properly.
- Check that the seat is well positioned in the lower frame before you go sitting in the swivel seat.
- Only Vermeiren approved accessories are allowed.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

11. Maintenance, inspection, cleaning

For the maintenance manual of the swivel seat refer to the Vermeiren website: www.vermeiren.be.

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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All rights reserved, including translation.
Vermeiren Suisse S.A.