User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Laly

EN

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Verze: B, 2022-11

All rights reserved, including translation.

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1 Content

Content	1
1 Preface	1
2 Your product	1
3 Before use	1
4 Assembly	2
5 Using your shower chair	2
6 Maintenance	3
7 Technical specifications	3

2 Preface

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the shower chair, read the instruction manual carefully: it will familiarise you with this product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.


Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

3 Your product

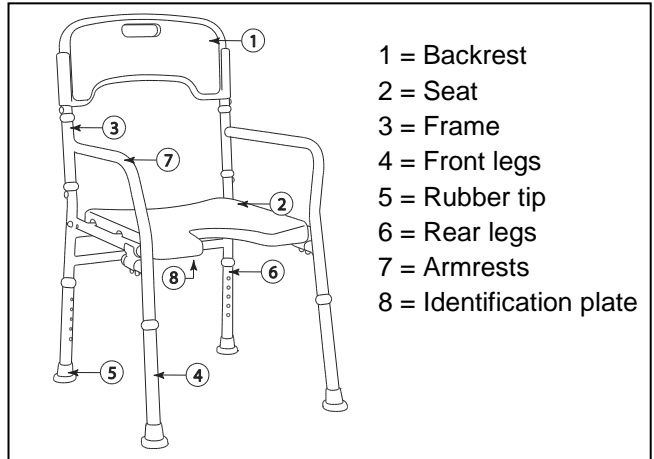4 Before use

4.1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Seat
- Backrest
- Height adjustment legs
- 4 Rubber tips
- Armrests
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

4.2 Intended use

- Indications and contraindication: The construction of the chair allows full use by elderly people or during revalidation in home environment. The shower chair shall be used by 1 person.
- The shower chair shall be used indoor, in the shower, bathroom.
- The chair shall not be used as toy for children.
- The chair should only be used on horizontal, non-smooth surfaces.
- The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

4.3 Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for your chair is 130 kg.**

- Use the shower chair on horizontal, non-smooth surfaces, all four legs must have contact with the ground.
- Keep it away from children. The shower chair shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Risk of burns – Be careful when using in extreme hot or cold environments for a sufficient amount of time and when touching.
- Do not use the shower chair when it is not working properly.
- Secure the seat firmly before using the shower chair.
- Check that the shower chair is completely unfold before using it.
- Do not lift the shower chair when the user is seated on the chair - Risk of injury or damage.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

4.4 Symbols on the shower chair

Maximum mass

CE conformity


Type designation

5 Assembly

5.1 Mounting / Removing backrest

- ⚠ **CAUTION: Risk of injury - Check that the backrest is firmly fixated.**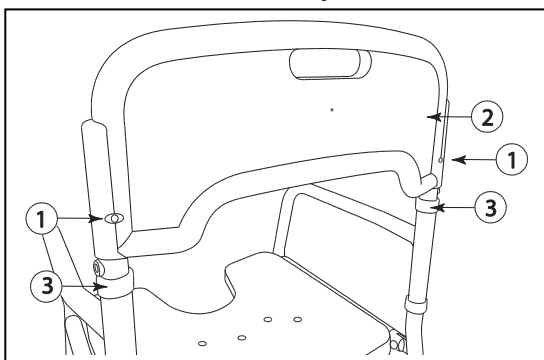

Mounting backrest:

- Mount the backrest ② on the two backrest tubes ③ until the spring pins ① clicks in the hole.
- Check that the backrest is firmly fixated.


Removing backrest:

- Press on the two spring pins ① on the rear of the backrest.
- Move the backrest ② upwards out the backrest tubes ③.

5.2 Folding / Unfolding the chair

- ⚠ **WARNING: Risk of injury - Check that the chair is completely unfolded before using it.**
- ⚠ **WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the entrapment zone.**

To transport the chair in the car or airplane you can fold the chair.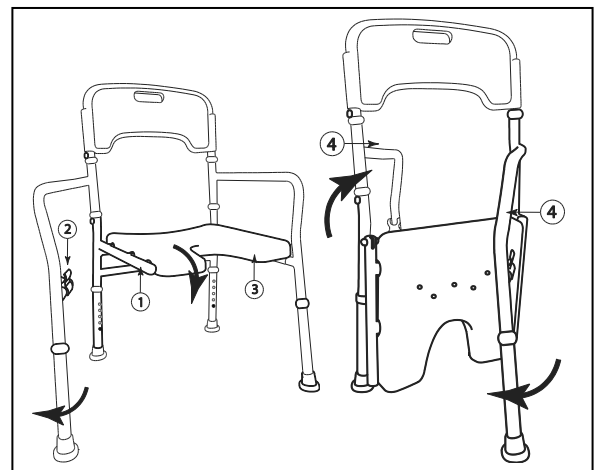

5.2.1 Folding shower chair

- Lift the seat tubes ① out the two plastic clips ②.
- Push the seat ③ downwards.
- Fold one side frame ④ frontwards against the seat.
- Fold the other side frame ④ backwards against the seat.

5.2.2 Unfolding shower chair

- Fold both side frames ④ to the side.
- Pull the seat ③ upwards.
- Push the seat tubes ① in the two plastic clips ②.


6 Using your shower chair

6.1 Seat height adjustment

- ⚠ **CAUTION: Risk of falling - Secure the seat firmly before using the shower chair. Check that the spring pin ① is firmly fixated in a hole.**
- ⚠ **CAUTION: Risk of falling - Do not exceed the maximum seat height**

adjustment, your shower chair will not be stable anymore.

⚠ CAUTION: Risk of tipping over - Place the four legs at the same height.

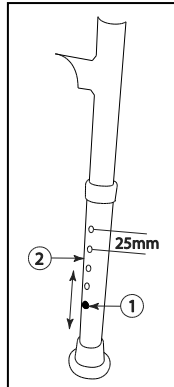Before using the shower chair you must adjust the seat in the correct height.

The seat can be adjusted in height by following steps:

- Press on the spring pin ①.
- Move the leg ② until the desired height (5 different positions: steps from 25 mm).
- Check that the spring pin ① is clicked back and firmly fixated in a hole.
- Adjust the other legs with the same instructions.
- Check that all legs are placed in the same position.

6.2 Carrying the shower chair

The best way to carry the shower chair: grasp the handgrip of the backrest or grasp the two armrests.

7 Maintenance

7.1 Cleaning and disinfection

Use a soft, damp cloth and lukewarm water with a mild household soap or disinfectant to clean the shower chair. Follow the instructions for use of the cleaning product. Avoid soaking. Do not use abrasive cleaning agents to clean.

When used in long-term care environments, follow the in-house instructions for cleaning and disinfection.

7.2 Inspection / Service / Repairs

The expected service life of your shower chair is 5 years. The lifespan is influenced by use, storage, maintenance, service and cleaning.

Before you resume use of the shower chair, it should be inspected on following points:

- Frame (deformation, stability, connections).

Your shower chair is constructed of aluminium and plastic components.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

7.3 Reuse

Before each reuse, have the shower chair disinfected, inspected and serviced according to the instructions in §7.1 and §7.2 .

7.4 Disposal at end of life

At end of life, you need to dispose of your shower chair according to the local environmental legislation.

8 Technical specifications

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Brand	Vermeiren
Type	Foldable shower chair
Model	Laly
Width	540 mm
Length	580 mm
Height	910 mm - 1010 mm
Dismantled width	520 mm
Dismantled length	790 mm
Dismantled height	120 mm
Seat height	430 mm - 530 mm
Seat width	410 mm
Seat depth	410 mm
Backrest height	500 mm
Weight total	3,80 kg
Patient weight	max. 130 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

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**Service registration form**

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com