



Lower frame
of Altitude2 or Coraille2

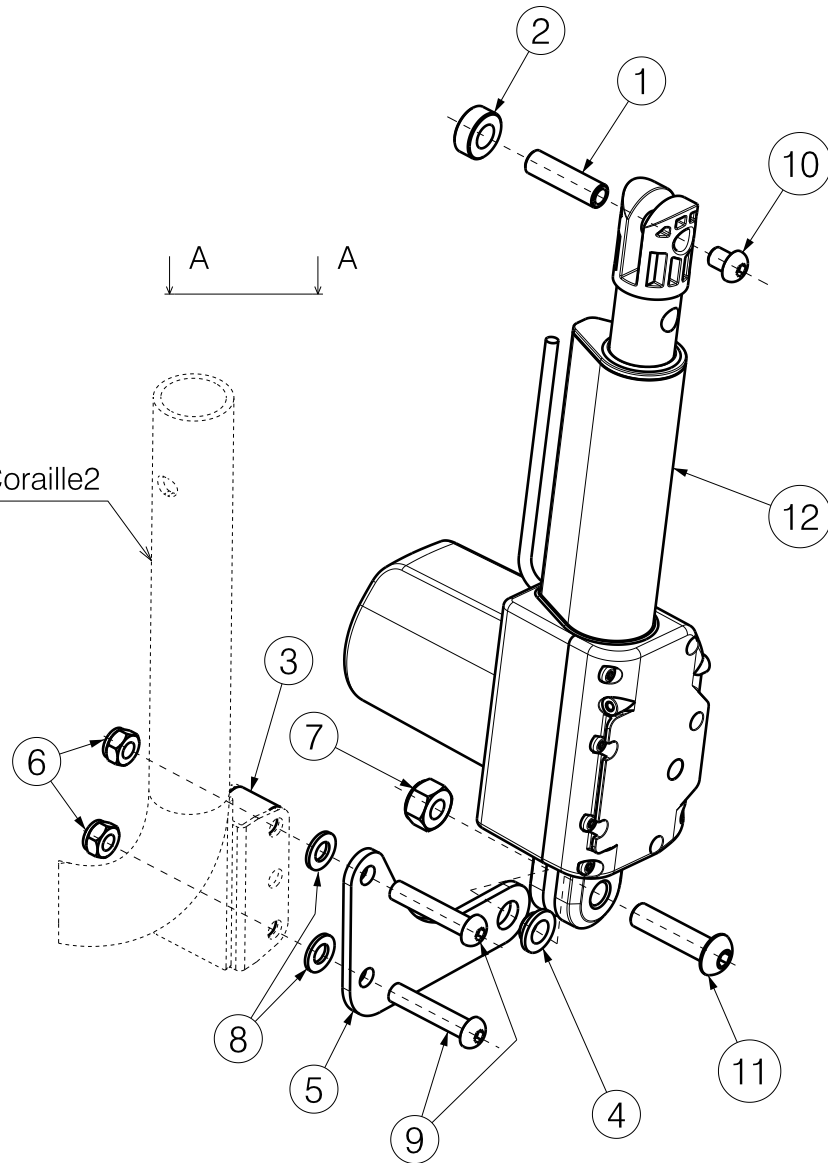


Material			
No.	Plan	Article	Quantity
1	X004804	1754800	1
2	X002029	1991841	1
3	XP00871	2830300	2
4	XP01931	2993584	1
5	YP06665	2732982	1
6	Nut M8	1001682	2
7	Nut M10	1001683	1
8	Washer M8	1003199	2
9	M8x40	1008739	2
10	M8x10	1992177	1
11	M10x40	1992592	1
12	Motor 3500N	2992973	1
13*	Power supply	2992974	1
14*	Remote control	2992975	1

* - not depicted / nie ujeta

Auxiliary view A
Rotated 180°

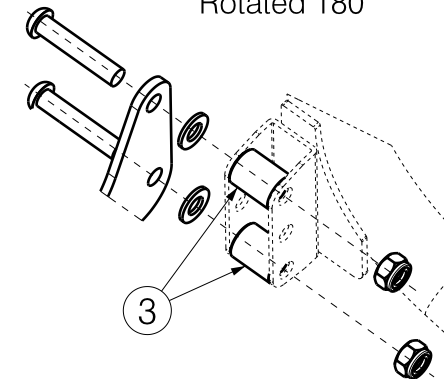

EP00333 / A

Electric kit for Altitude 2 & Coraille 2				Plan No.
				EP00333/ A
Scale: 1:3	Drafted	Verified	Confirmed	
Date	14/09/2018	14/09/2018	14/09/2018	
Initials	DR	DB	TK	