

Material			
No.	Name	Article	Quantity
1	Belt 40mm	1009517	A
2	Belt 50mm	1009520	A
3	Velcro hook 50mm	1009791	0,67m
4	Velcro hook 50mm	1009791	0,72m
5	Velcro hook 40mm	1905877	0,18m
6	Velcro 50mm	1009792	0,72m
7	Velcro 40mm	1905876	0,18m
8	Belt 50mm	1009520	0,67m
9	Buckle 40mm	1009512	1
10	Buckle 50mm	1009511	4
11	Belt 50mm	1009520	C
12	Velcro hook 50mm	1009791	D
13	Belt 20mm	1009515	0,21m
14	Belt 20mm	1009515	0,888m
15	Velcro 40mm	1905876	0,41m

Complete 5-belts L44 backrest	
Size	Article
35	1921136
40	1915717
45	1915718
50	1915719


BACKREST BELTS L44 INOVYS				Page 1/2	Plan No.
				EP00494 / B	
Scale: 1:5	Drafted	Verified	Confirmed		
Date	19/02/2020	06/03/2024	06/03/2024		
Initials	AS	AZ	AS		

EP00494 / B




Prepere Materials:


1- 1009517 Belt

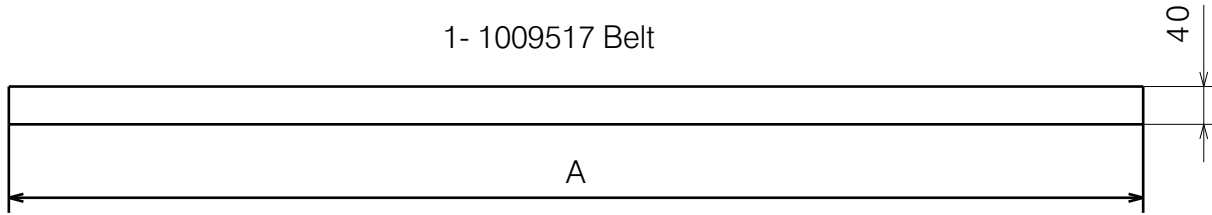


2-1009520 Belt

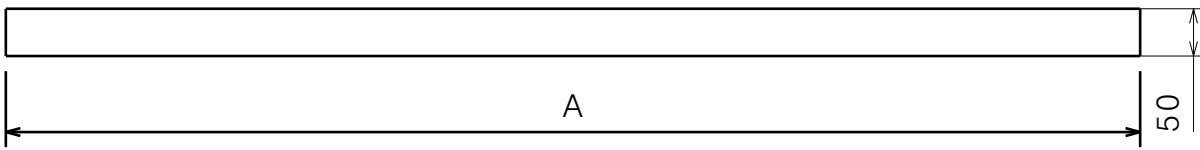


6-1009792 Velcro

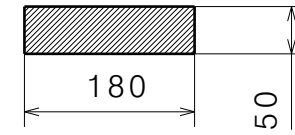


7-1905876 Velcro

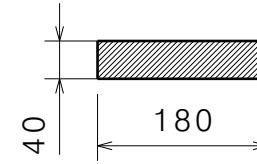


8-1009520 Belt

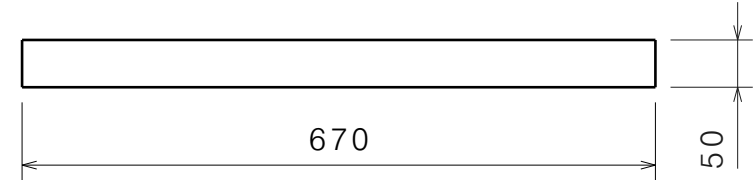


3- 1009791 Velcro hook

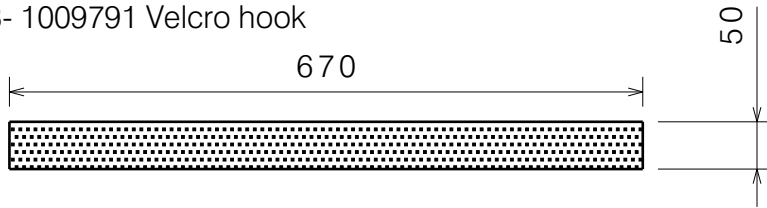


4-1009791 Velcro hook

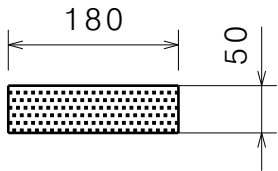


5-1905877 Velcro hook

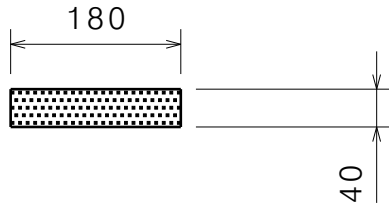


11- 1009520 Belt

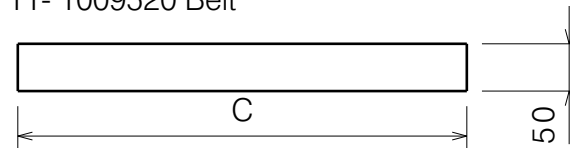


12- 1009791 Velcro

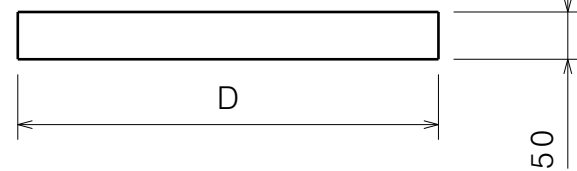


13-1009515 Belt

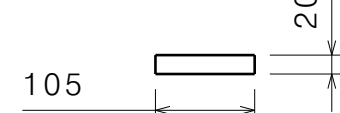

14-1009515 Belt

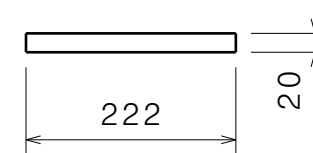

15- 1905877 Velcro hook 40mm cutted half

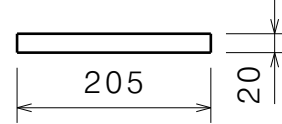

Type	35	40	45	50
A	1000	1100	1200	1300
B	160	210	260	310
C	425	475	525	575
D	395	445	495	545

BACKREST BELTS L44 INOVYS

Page 2/2

Plan No.

EP00494 / B

Scale: 1:5

Drafted

Verified

Confirmed

Date

19/02/2020

06/03/2024

06/03/2024

Initials

AS

AZ

AS



EP00494 / B